

START
HERE!

DIP⁷⁰


Instructions for use.
Sampling kit 2 x 70µl.

For research use only, not for
use in diagnostic procedures.

 Capitainer®

INCLUDED IN THE SAMPLING KIT.

Single use products.

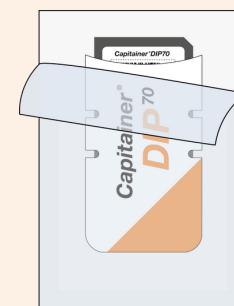

◀ (a) Capitainer®DIP70
sampling card


◀ (b) Capitainer®DIP70
Drying pouch



(c)
Sample ID



(d)
Sample
container

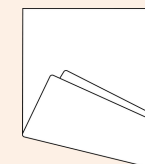

(e)
Absorbent
paper

Additional items you will need:

- A sample ID provided by your kit provider
- Instructions for how to return your sample to the laboratory

01

PREPARE

1



- Read entire instruction before you start
- Scan QR code or enter <https://capitainer.com/sampling-video-dip70/> to access video instructions

2




- Wash hands with soap and warm water. Dry fully with a clean towel
- Choose a place where you can stand or sit comfortably
- Place all material on flat clean surface

3



- Open sampling card package
- Take out sampling card and drying pouch

4

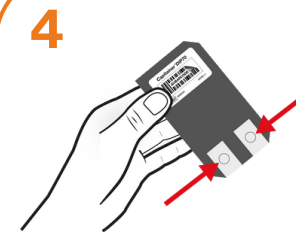

- Place sampling card on flat clean surface
- Keep drying pouch nearby

 **Avoid touching
sampling discs or the
sample cavities!**

5



- Label sampling card with sample ID (c)

 Continue on
page two

02


SAMPLE



Warnings and cautions


- Do not touch the sampling discs or the sample cavities
- Use the card within 30 minutes of opening the package
- Ensure to collect enough urine in the sample container so the sample cavities can be fully immersed before dipping

1

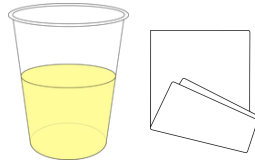

- Have the sample container (d) and the absorbent paper (e) available
- Collect urine in the sample container (d)

2

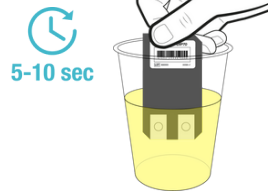

- Hold the sampling card upright and immerse the sampling discs for **5-10 seconds** in the urine, just until the cavities are fully covered



Do not go past the cavities

3




- Remove sampling card away from the urine sample
- Keep the card upright and tap the dipped end to the absorbent paper (e) **three times** firmly until most excess liquid is gone

03

PACK & RETURN

All used materials must be handled and disposed of according to local regulations

1

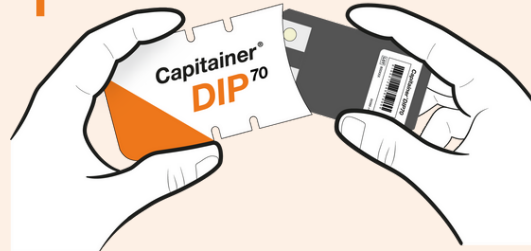

- Put the card back into the drying pouch (b)
- Close inlet flap
- Return sample as directed by your test provider

Thank you for following the instructions for sampling with Capitainer®DIP70. If you have questions, please contact your test provider.

 **Capitainer®**